

Limited Warranty

ARTISAN of CANADA warrants the frame of your steel / aluminum four wheeled walker against defects in materials and workmanship for a period of twenty four (24) months.

ARTISAN of CANADA distributed components (including brakes, wheels and basket) are warrantied against defects in materials and workmanship for one (1) year from the date of purchase. Items considered normal wear and tear (such as tires, tubes, upholstery, cables and handgrips) are not covered by this warranty.

ARTISAN of CANADA will repair or replace parts as required, provided that authorization for the return has been given to an authorized Artisan Dealer, within the appropriate warranty period.

Labour charges and freight charges are not included in the warranty coverage.

Any components damaged by improper installation, neglect, misuse, abuse or being serviced by someone other than an Artisan Dealer, will not be warrantied.

ARTISAN of CANADA sole liability extends to repair or replacement of parts or components as previously specified. The forgoing is in lieu of all warranties, expressed, implied or statutory, whether written or oral.

This warranty is applicable in Canada only.

WARNING! Read Before Using

This walker is not intended to be used as a wheelchair. For maximum safety, make sure all safety features are explained to you by a qualified therapist, physician or a certified sales person. At regular intervals, have your local dealer check and tighten all fasteners and hinged parts. Check all plastic parts often for wear. When stopping, do not lean excessively on the unit. The brakes are not designed to withstand excessive pressure.

Excessive use causes the brake cables to stretch and fray. Adjustment or replacement of the brake cable may be required. Artisan of Canada walkers are not suggested for people with severely limited ambulation for whom a rigid walker may be recommended. Please consult with your physician before using this product, especially if you are taking any medication.

Purchase Date: _____

Dealer: _____

Serial Number: _____

Dealer Phone: _____

Made in China

rev.02.25.20

EN

ARTISAN of CANADA
Division of BDARK Holdings Ltd.

OVATION

805

**Compact
Soft Seat**

Instructions for Tool Free Assembly

WARNING! It is very important that you read all instructions before assembly and/or use of this walker. Before using the walker, be sure your weight is within the weight capacity of the unit. This walker will require little assembly, which can be performed without tools.

Parts of your Walker

- 1 Hand Grips
- 2 Hand Brake
- 3 Large Foldable Adjustable Backrest
- 4 Brake Cable in Frame
- 5 Push Button for Handle Adjustment
- 6 Curb Assist Climber
- 7 Seat Sling
- 8 Foldable Cloth Basket
- 9 Reflective Strip in Front Forks
- 10 Cane Holder
- 11 Front Wheels & Fork
- 12 Rear Wheels
- 13 Pull Strap to Fold



www.ovationwalker.com

BEFORE USE, PLEASE READ ALL CAUTION AND WARNING LABELS ON THE WALKER

WALKING

This walker is to be used as a mobility aid creating a natural balance. With hands grasped securely onto handles, slowly walk behind the walker. Gently push forward using evenly distributed weight between the handle bars.

BRAKING

Normal Brake can be activated by squeezing the brake handle similar to operating the brakes of a bicycle. This function may be applied while walking with the walker to slow down momentum, to stop for a moment, or to manoeuvre up, down or around obstacles which may be in your path.

Parking Brake can be activated by pushing the brake handle down into the lock position. This function should be activated when the user stops for a period of time. For safety reasons, the parking brake should always be engaged before sitting down onto the seat. For example, while shopping, it is beneficial to activate the parking brake while stopped in an aisle to avoid the walker from advancing forward.

To release the parking brake, gently pull up.


The braking system is convenient and easy to operate. People who may have hand disabilities should find the brake handles wide, comfortable and non-binding. Minimum pressure needs to be applied to activate the locking loop brake system.



Squeeze to Stop



Release to Roll

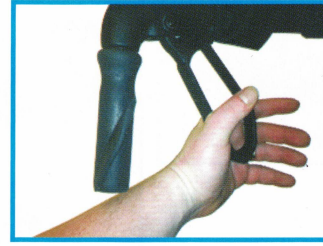

Push Down to Lock

UNFOLDING

To unfold your Ovation Compact walker, carefully push down on both sides of the seat with your hands flat until you hear a click. Do not wrap your fingers around seat as they may get pinched between seat & frame.

FOLDING

From the user side of the walker, lift the safety strap and pull up to release the safety latches, then lift into the folded position. The walker will fold easily for storage and transport.

The cloth basket on the Ovation Compact model folds with the walker.

ADJUSTING HANDLE HEIGHT

To adjust the handle height, locate the red push button on either side of the walker (located where handle goes into the walker). Push the red button in. While holding button in, adjust handles to required height (the heights are marked on the inside of handle post from 1-6). Repeat steps with other side of walker. Match the number to the first handle. Always adjust both sides the same height.

ADJUSTING BACK STRAP

The back strap of the Ovation Compact walker can be adjusted if need be. To adjust back strap just loosen red knobs (located just under brake handle) and slide strap to desired place. Then tighten red knobs up.

PLACEMENT OF CANE HOLDER

The cane holder on the Ovation Compact walker can be placed on the left or right side of walker. The cane clip in the handle just slides up and off, you can then place it on the desired handle. The cup for the cane to sit in just clips off. Grasp the bottom clip and pull up to pop it off. Then clip to desired side.

MAINTENANCE

Always make sure your brakes are operating correctly before use. Keep the brake handles free from dirt, dust and grime. Build up of particles could cause your brakes to engage improperly. If the brakes are not functioning, please take your walker to an authorized Artisan of Canada Dealer to have them service the brakes. If the cable is broken or stretched, it will need to be replaced. **Periodically check and tighten all nuts, bolts, screws and other hardware.**

REMEMBER! Always activate the parking brake **before** sitting.

CAUTION! Check brakes **before** using.